

Headrest Mounting Kit

User Instructions

August 2011

Introduction to the Headrest Mounting Kit



The Headrest Mounting Kit enables the use of either the height adjustable headrest or the height and depth adjustable headrest to be used with the M2 range of Shower, Toilet and Commode chairs.

Please note the headrests and their associated mounting kits are provided as standard with any M2 Tip chairs.

Questions and Concerns

Should you experience any difficulties or have any concerns regarding the initial purchase or subsequently adjusting the Headrest Mounting Kit or have any concerns regarding its use or operation then immediately contact either your Kirton representative, distributor or our Customer Service Team on **Freephone 0800 212709 or +44 (0) 1440 705352**, they will be delighted to help you.

Guidelines for Best Practice

To ensure good practice when considering purchasing a Headrest Mounting Kit and effective post purchase we advise that the following guidelines are considered:

1. Seek advice and input from a qualified therapist/professional and/or Kirton representative/distributor at the initial prescription of the chair.
2. If unsure when ordering the chair and any accessories contact either the therapist involved or your Kirton representative, distributor or our Customer Service Team on **Freephone 0800 212709 or +44 (0) 1440 705352**, to discuss further.
3. Users of the Headrest Mounting Kit, where necessary, should be supervised to ensure the most appropriate position is maintained in the chair.

The User Instructions

The purpose of the following user instructions is to provide a guide on how to fix the Headrest Mounting Kit to the chair.

The instructions should be read by all those involved with the supervision of the user and/or the care of the chair and its accessories before use.

Please retain these instructions with the Care and Maintenance card enclosed for future reference.


Attaching the Headrest Mounting Kit Bracket




The headrest mounting kit is a bracket that is attached to the backrest of the chair.

Remove the four original screws in the backrest of the chair using an Alun key. Position the bracket over the holes and fix in place using the four longer screws provided.

Positioning the Height and Depth Adjustable Headrest

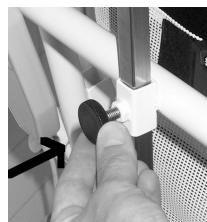

To adjust the height, loosen the thumbscrew on the holder at the base of the headrest, reposition and tighten.

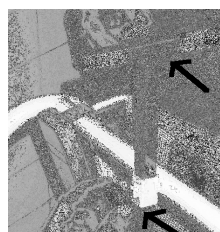

To adjust the depth, loosen the thumbscrew at the top of the support arm and move the headrest in/out to the desired distance. Retighten the thumbscrew.




The wings of the headrest can be moved in and out to support the head. Apply pressure to either side of the headrest and push inwards to reduce the interior size of the headrest or alternatively position your hands on the inside of the headrest and pull out to increase the interior size of the headrest. **DO NOT APPLY PRESSURE TO THE TWO ENDS OF THE HEADREST AS IT WILL BREAK THE EDGES.**

Headrest Mounting Kit

User Instructions

August 2011

Positioning the Height Adjustable Headrest

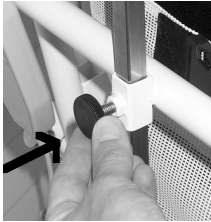

To adjust the height, loosen the thumbscrew on the holder at the base of the headrest, reposition and retighten.



The wings of the headrest can be moved in and out to support the head. Apply pressure to either side of the headrest and push inwards to reduce the interior size of the headrest or alternatively position your hands on the

inside of the headrest and pull out to increase the interior size of the headrest. **DO NOT APPLY PRESSURE TO THE TWO ENDS OF THE HEADREST AS IT WILL BREAK THE EDGES.**

Helpline

Should you have any concerns either pre or post purchase of the chair contact our Customer Service Team on **Freephone 0800 212709 or +44 (0) 1440 705352**, to discuss further. Alternatively, if you purchased your chair from a Kirton distributor in the first instance please contact your point of reference there.

Potential Risks, Cautions and Warnings

- The user is at risk if the Headrest Mounting Kit or headrests are not properly set up to their requirements.
- When attaching the Headrest Mounting Kit ensure that the screws are tight.
- Ensure the headrest is appropriately positioned to support the head of the user.
- Regularly check the user to ensure the appropriate position is maintained.

Care and Maintenance

- Wipe down the framework with soapy water and a soft cloth to reduce the build up of soap and limescale.
- Cleaning with a scouring pad is not advised.
- Maximum temperature 80 °c.



23 Rookwood Way
Haverhill
Suffolk, CB9 8PB
England

Telephone: +44 (0) 1440 705352
Fax: +44 (0) 1440 706199
Freephone: 0800 212709

Email: info@kirtonhealthcare.co.uk
www.kirton-healthcare.co.uk

